**PURPOSE**

The LB78 Dump Valve enables exhaled gases to be expelled at an ambient pressure level lower than that of breathing. It is used specially for evacuating to the outside of an hyperbaric chamber the oxygen rich mixtures breathed during decompression or therapeutic treatments.

This prevents oxygen build-up in the atmosphere of the chamber, thus minimising fire risks.

**DESCRIPTION**

The overpressure created by the expiration operates a diaphragm which, in turn, opens a large diameter fully balanced exhaust valve.

A large gas flow is thus obtainable with a minimum driving overpressure.

It is equipped with a safety valve which prevents any dangerous pressure-drop in the breathing circuit, in the event of the main valve leaking.

The LB 78 oxygen dump-valve features a circular housing made of medical POM-C easy to clean.

The stainless steel threaded locking ring gives quick access to the diaphragm to ease cleaning and periodic maintenance.

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**CHARACTERISTICS**

- Individual breathing equipment
- Fully rated for use with pure oxygen under pressure
- Operating differential pressure : ..... 0.1 to 5 bar  
*For higher pressures, a second stage dump valve ref. 422 01028 must be added to the circuit.*
- Dump valve opening (overpressure) : ..... 2 millibar
- Maximum instant flow rate : ..... > 400 l/min
- Safety valve opening (underpressure) : ..... -2 millibar
- LB78 Dump valve inlet is a dia. 22 mm straight hose connector that may easily be adapted to any LP hoses
- Housing made of medical POM-C and Stainless Steel
- Mechanism of chrome plated brass and Stainless Steel
- Diaphragm made of silicone
- O' ring sealing
- Simple and rugged, no adjustment needed

**DIMENSIONS**

- Weight : ..... 900 g
- Diameter : ..... 127 mm
- Height : ..... 65 mm
- Delivered packed in a 22 x 22 x 10 cm card-board box

**Corrugated Hose (sold separately):**

- Heavy-duty LP Hose to connect the dump valve to the oronasal mask
- Made of Medical Silicone
- Length : ..... 800 mm

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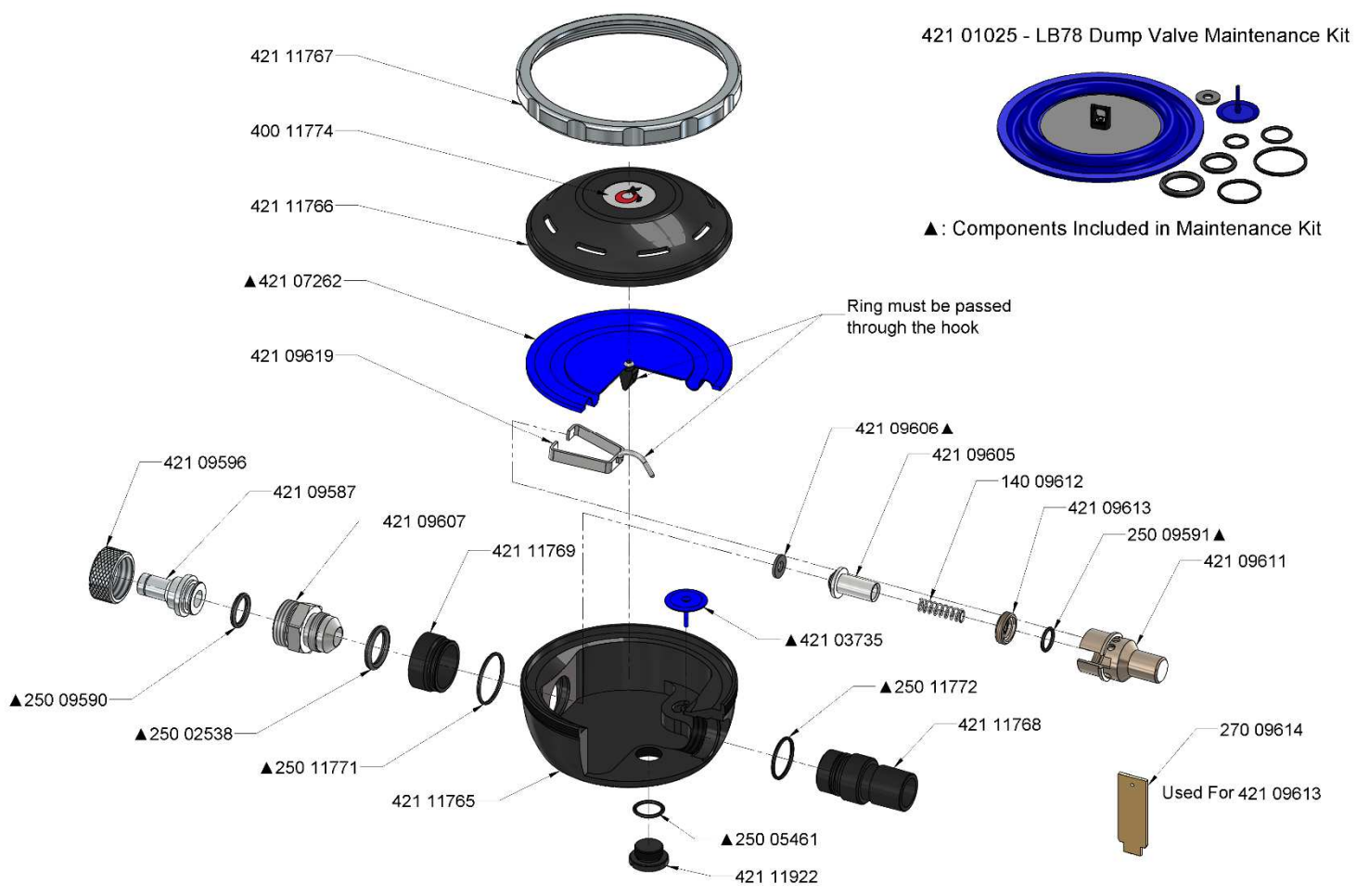
**MAINTENANCE**

- Thanks to its design, the LB78 dump valve requires no adjustment.
- For optimal functioning, it should be cleaned every 6 months and the O'rings should be slightly lubricated with an inflammable liquid grease.
- All soft parts should be changed every year (maintenance kit 421 01025).
- The tightness of the main valve can be checked by blowing into or out of outlet 421 09587.

**DISMANTLING**

- Unscrew the ring 421 11767.
- Take off cover 421 11766.
- Take out diaphragm 421 07262 by sliding it to free the ring from lever 421 09619.
- With a 27mm spanner, undo stainless steel seating 421 09607, thus enabling valve body 421 09611 to be disengaged.
- The two moving parts can then be removed: lever 421 09619 and piston 421 09605, as well as spring 140 09612.
- O'ring 250 09591 can be taken out by undoing the seal retainer 421 09613 by means of the special tool 270 09614 supplied with the dump valve.
- Pull on the outlet pipe 421 11769 to access the seal 250 11771.
- Unscrew the inlet pipe 421 11768 to access the seal 250 11772.

**CLEANING**

- Clean all parts in soapy water.
- Rinse thoroughly. Dry using clean and fresh air.
- Lubricate slightly piston 421 09605 and O'ring 250 09591 using an inflammable liquid grease.

**In the event of a leak,**

- Check O'rings 250 09590, 250 02538, 250 29591, 250 1177 and 250 11772.
- Check valve seal 421 09606, and clean it carefully.
- Check the stainless steel seating 421 09607, inspect with particular care the edge of the seating which shall not be scratched. In the event of a scratch on the seating, polish it on an oilstone.
- Check piston 421 09605. If scratched, it must be replaced.

**The dump valve is hard to blow through,**

- Clean and lubricate it.
- Check carefully piston 421 09605 and its seal 250 09591.
- Check diaphragm 421 07262 which may have lost its flexibility, in which case, it should be changed.
- Check spring 140 09612.

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