**PURPOSE**

This unit enables to inhale a breathing mixture which is different from the one constituting the ambient atmosphere.

It has been specially designed to breathe oxygen-rich mixtures used in hyperbaric oxygen therapy or by divers during chamber decompression.

This is particularly interesting for those who prefer to use disposable hoses and masks rather than our standard heavy-duty corrugated hose and mask.

DESCRIPTION

The Wall-mounted Demand Regulator is connected directly to the gas supply entering the chamber.

Only a LP hose may then be connected to the breathing mask which is much lighter and comfortable than the usual BIBS mask.

The regulator balanced demand-valve can supply a large gas-flow for a minimum inhaling effort.

The Wall-Mounted Demand Regulator may be used with especially designed COMANEX Breathing Mask and LB 78 Dump Valve or any other appropriate pieces of equipment.

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CHARACTERISTICS

- Individual breathing equipment
- Fully rated for use with pure oxygen under pressure
- Gas Supply relative pressure : 2 to 8 bar
- Valve opening (overpressure) : 2 millibar
- Maximum instant flow rate : > 400 l/min
for a 5 millibar depression
- Demand regulator Inlet : 1/4" BSP Female
- Regulator outlet is a dia. 22 mm straight hose connector that may easily be adapted to any LP hoses
- Housing made of medical POM-C and Stainless Steel
- Mechanism made of Stainless Steel
- Diaphragm made of silicone
- O' ring sealing
- Simple and rugged, no adjustment needed

DIMENSIONS

- Weight : 920 g
- Diameter : 127 mm
- Height : 66 mm
- Delivered packed in a 22 x 22 x 10 cm card-board box

Corrugated Hose (sold separately):

- Heavy-duty LP Hose to connect the Demand Regulator to the Breathing Mask
- Made of Medical Silicone
- Length : 800 mm

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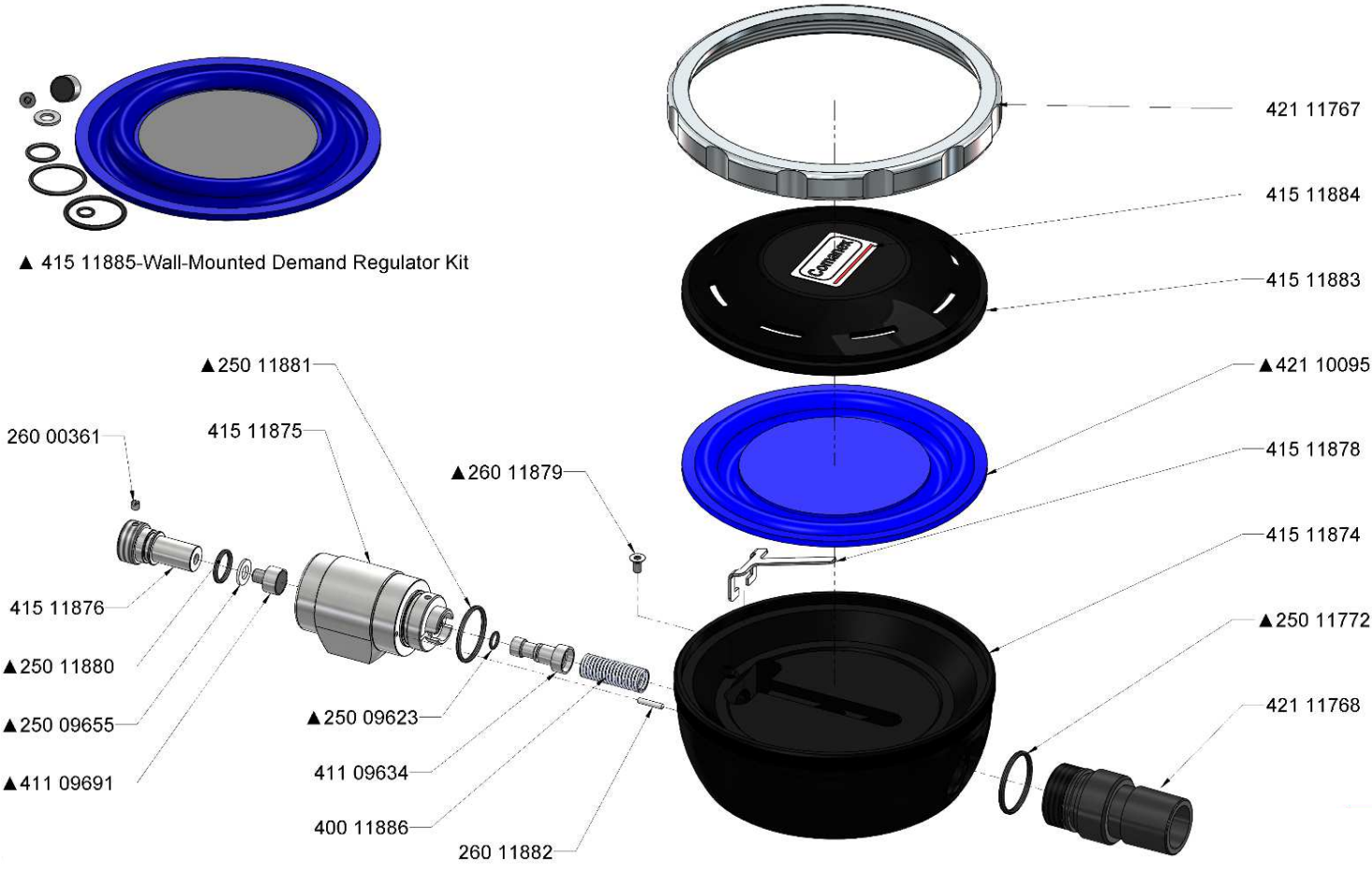
MAINTENANCE

- Thanks to its design, the wall-mounted demand regulator requires no adjustment.
- For optimal functioning, it should be cleaned every 6 months and the O'rings should be slightly lubricated with an inflammable liquid grease.
- All soft parts should be changed every year (maintenance kit 415 11885).

DISMANTLING

- Unscrew the ring 421 11767.
- Take off cover 415 11883.
- Take out diaphragm 421 10095.
- With a 6 mm Allen wrench, unscrew the seat holder 415 11876 to access the nozzle 411 09634.
- Disengage lever 415 11878 by pushing the nozzle 411 09634 with a plastic tool in order not to damage the nozzle edge.
- With a 2 mm Allen wrench, remove screw 260 11879.
- Uncouple inlet body 415 11875 gently in order to unload spring 400 11886.
- Nozzle 411 09634 may now be removed from inlet body 415 11875.
- Unscrew seat 411 09691 from seat holder 415 11876 using soft pliers.
- Unscrew the pipe fitting 421 11768 to access the seal 250 11772.

CLEANING

- Clean all parts in soapy water.
- Rinse thoroughly. Dry using clean and fresh air.
- Lubricate slightly nozzle 411 09634 and O'rings using an inflammable liquid grease.

ASSEMBLING AND SETTING

- Assembling in reverse order to the dismantling.
- At the end of the assembling, screw the seat holder 415 11876 in the inlet body 415 11875 until the lever 415 11878 detaches from the main body 415 11874 (get a movement of +/- 1 mm at the tip of the lever).

In the event of a leak,

- Check movement of +/- 1 mm at the tip of the lever.
- Check all O'rings.
- Check vulcanised seat 411 09691.
- Check stainless steel nozzle 411 09634, inspect with particular care the edge which shall not be scratched.
- Check surface condition of inlet body 415 11875 where the nozzle 411 09634 translates.

The demand regulator hard to blow through,

- Clean and lubricate it.
- Check movement of +/- 1 mm at the tip of the lever.
- Check diaphragm 421 10095 which may have lost its flexibility, in which case, it should be changed.
- Check spring 400 11886.

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